# INSTRUCTIONS

## BOXER'S FRACTURE BRACE

antimicrobial treated\*



**BOXER'S FRACTURE BRACE (BFB)**  
**ORTESIS PARA FRACTURA DEL BOXEADOR**  
**EXOS™ HAND- / UNTERARMORTHESE MIT 4./5. FINGERFIXIERUNG MIT BOA®**  
**TUTORE PER FRATTURA DEL PUGILE**  
**ATTELLE POUR FRACTURE DU BOXEUR**  
**BOKSER FRACTURE ORTHESE**  
**ORTÉZA NA BOXERSKÉ ZLOMENINY**  
**ORTÉZA PRO BOXERSKÉ ZLOMENINY**  
**APOIO PARA FRACTURA DE PUGILISTA**  
**拳击手骨折护具**  
**ボクサー骨折用ブレース**

**DJO GLOBAL**  
 Distributed by: DJO, LLC  
 1430 Decision Street,  
 Vista, CA 92081  
 P 800.336.6569  
 F 800.336.6569  
 djo@global.com

no. 10020 Rev G  
 USA and International Patents Pending

**CE** **REP**  
 Exos™ Corporation  
 1122 Red Fox Road  
 Arden Hills, MN 55112  
 Medical Device & QA Services Ltd  
 Timperley, Cheshire,  
 WA15 7SN United Kingdom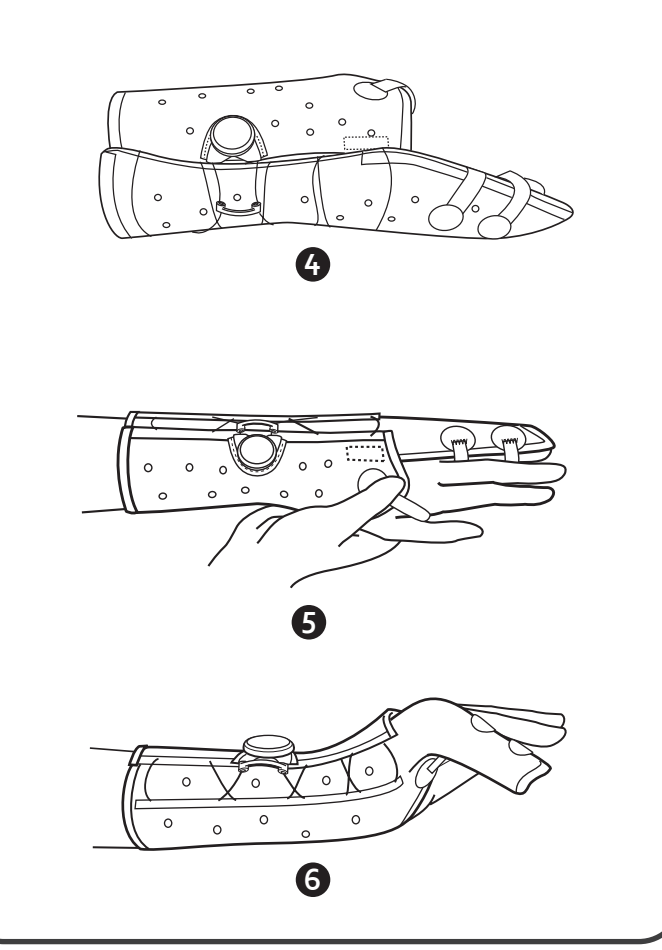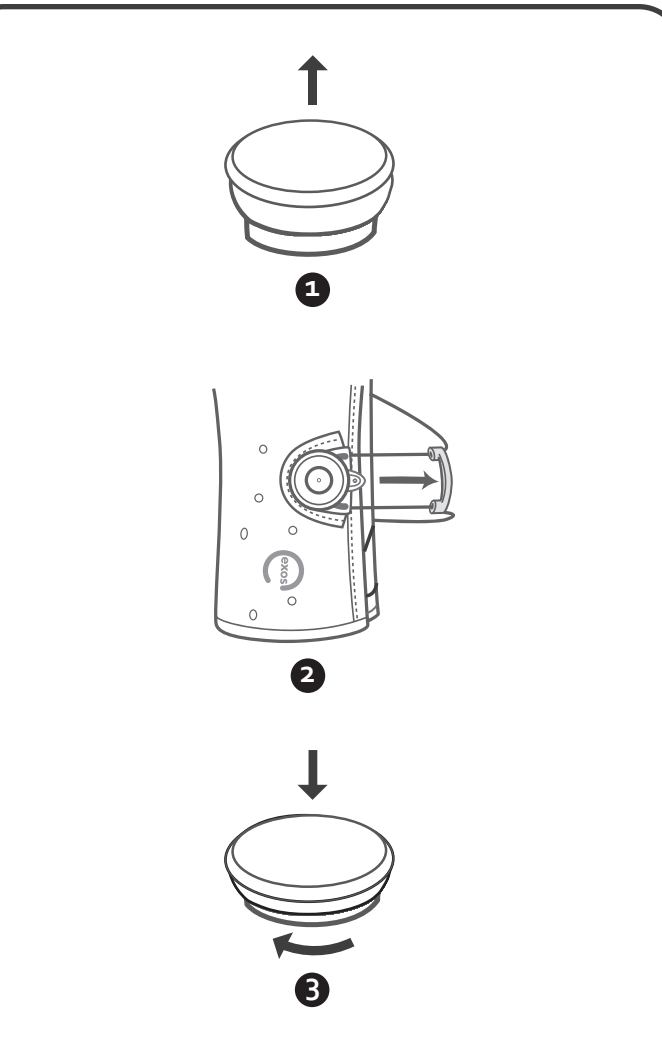

### ENGLISH

Before using the device, please read the entire instruction manual. Correct application and care is vital to the proper functioning of the device.

**INTENDED USE / INDICATIONS**  
 The Boxer's Fracture Brace (BFB) provides stabilization to the fourth and fifth metacarpals, the fourth and fifth digits, and the wrist joint.

**CONTRAINDICATIONS**  
 Patients with known allergies to foams, adhesives, or plastic products.

- ⚠ WARNINGS and PRECAUTIONS**
- ▶ Brace should be fitted by a qualified healthcare professional.
  - ▶ Do not heat brace on the body.
  - ▶ Check to make sure brace is not too hot prior to application.
  - ▶ Brace should be appropriately sized to the patient to ensure proper support and function.
  - ▶ Care should be taken with patients with delicate or compromised skin, especially the elderly, not to apply the brace too tightly, which may CAUSE pressure and shear to the fragile tissue. PATIENTS SHOULD BE INSTRUCTED ON APPROPRIATE TENSIONING DURING ONGOING USE. An under sleeve or STOKKINETTE may be used for added protection.
  - ▶ Do not heat Exos™ products to greater than 212°F (100°C).
  - ▶ FOR SINGLE PATIENT USE ONLY.

**OPERATING BOA® LACING SYSTEM**

**To Loosen:**

- ▶ Lift Boa® Knob. (Figure 1)
- ▶ Pull Tab. (Figure 2)

**To Close and Tighten:**

- ▶ Push Boa® Knob down. (Figure 3)
- ▶ Turn Boa® Knob clockwise. (Figure 3)
- ▶ Boa® Knob must be in the down locked position when wearing brace.

**HEATING AND APPLICATION**

**▶ Open and unfold brace. (Figure 4)**

**Note: Larger braces are packaged with the finger sections folded lengthwise. Do not unfold fingers until the brace has been fully heated.**

- ▶ Proper heating temperature is 200°F (93°C).
- ▶ Heat brace in Exos recommended oven approximately 5 minutes or until brace is pliable. Brace should not be too hot to handle comfortably.
- ▶ Ensure that skin is completely dry before application.
- ▶ Fit brace on patient. (Figure 5)
- ▶ Remove slack in lace by tightening Boa® Knob.

**Caution: Do not tighten brace fully until brace has returned to room temperature otherwise indentations in plastic may occur.**

- ▶ Mold brace into desired position. (Figure 6)
- ▶ Hook-and-loop fastener straps should be secured in the desired position.
- ▶ Once brace is hardened, the Boa® Knob may be tightened to desired tension.

**Warning: The brace should not be worn too tightly. This may produce excessive mechanical force or shear forces to the skin.**

**CLEANING INSTRUCTIONS**

1. Loosen or remove the brace.
2. Wash inside and outside of brace and skin with antibacterial soap and water (86°F / 30°C).
3. Rinse thoroughly with water.
4. Use a hair dryer set on high volume and cool setting to thoroughly dry the interior of the brace, and skin.
5. Retighten the brace.

**For fitting tips on each Exos™ device:**  
 Visit the Exos™ Provider Center at:  
<http://exosmedical.com/provider/BFB> or  
 Scan the QR image with your smartphone

**LIMITED PRODUCT WARRANTY**  
 Exos™ covers defects in workmanship and materials up to 8 weeks from date this product is dispensed. Damage caused by normal wear and tear, abuse, or alterations is not covered.

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