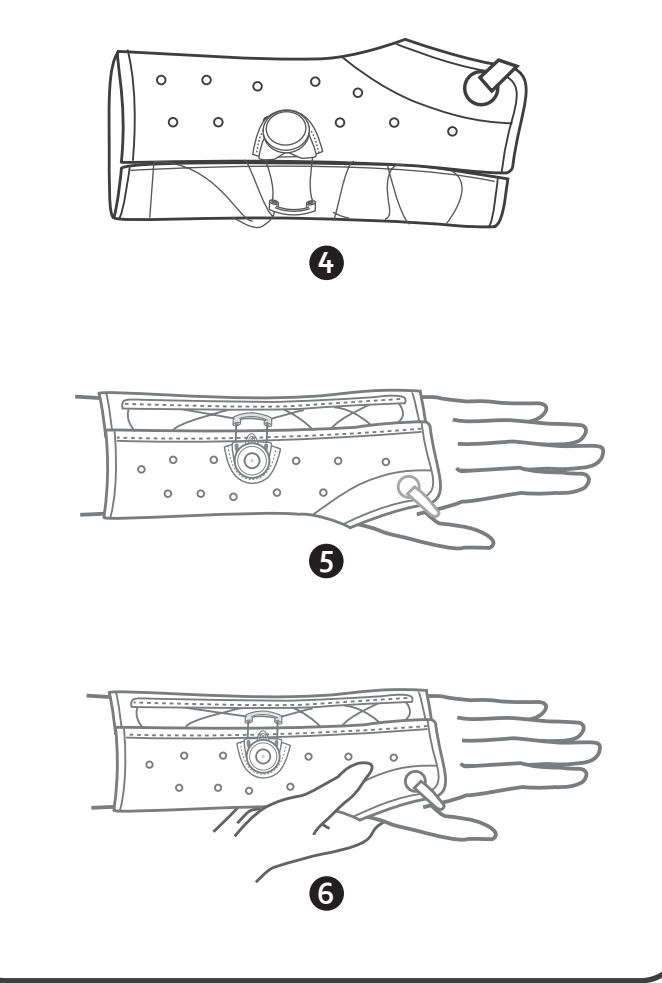
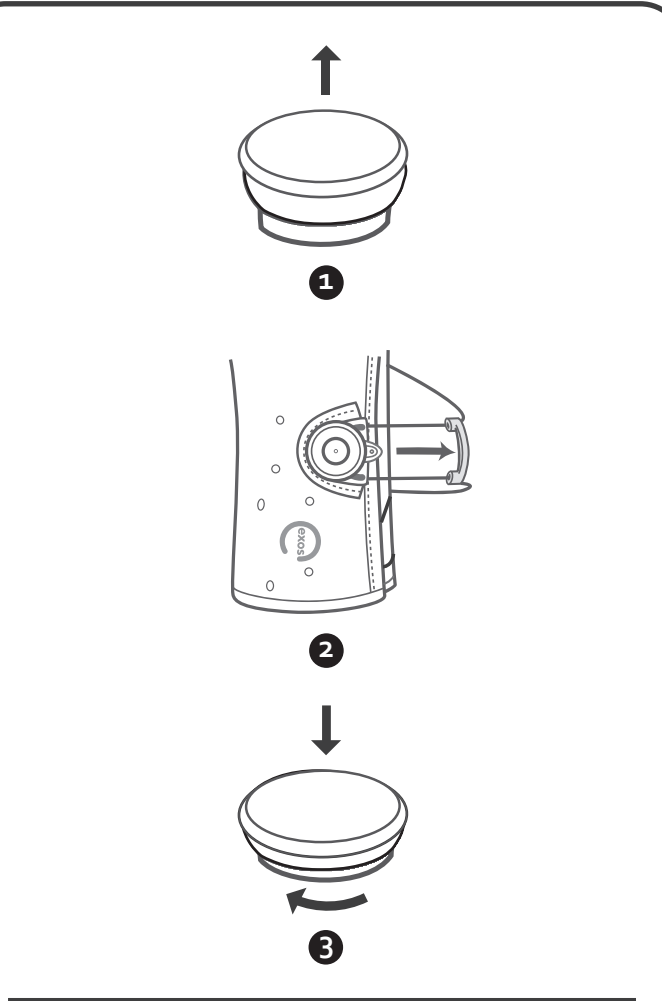
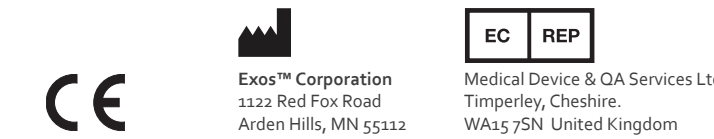
SHORT ARM FRACTURE BRACE OPEN THUMB (SAFB-OT)
ORTESIS CORTA PARA FRACTURAS DEL BRAZO CON PULGAR ABIERTO
EXOS™ HAND- / UNTERARMORHESSE MIT BOA®, OFFEN
TUTORE PER FRATTURA DELL'AVAMBRACCIO CON POLLICE APERTO
ATTELLE COURTE POUR FRACTURE DE BRAS À POUCE OUVERT
KORTE ARM FRACTUUR BRACE MET OPEN DUIM
KRÁTKÁ ORTÉZA RUKY NA ZLOMENINY S OTVOROM NA PALEC
KRÁTKÁ ORTÉZA PRO FRAKTURU PAŽE S VOLNÝM PALCEM
APOIO A CURTO PARA FRACTURA DO BRAÇO COM POLEGAR ABERTO
裸露拇指短臂骨折器具
ショート型腕骨折用ブレース (自在に親指を動かせるタイプ)

EXOS™ GLOBAL
 Distributed by: DJO, LLC
 1430 Decison Street,
 Vista, CA 92081
 P 800.336.6569
 F 800.336.6569
 djo@global.com

no. 10094, Rev D
 USA and International Patents Pending

ENGLISH

Before using the device, please read the entire instruction manual. Correct application and care is vital to the proper functioning of the device.

INTENDED USE / INDICATIONS
 The Short Arm Fracture Brace Open Thumb (SAFB-OT) provides stabilization to the distal radius, distal ulna, and wrist joint.

CONTRAINDICATIONS
 Patients with known allergies to foams, adhesives, or plastic products.

⚠️ WARNINGS and PRECAUTIONS
 ▶ Brace should be fitted by a qualified healthcare professional.
 ▶ Do not heat brace on the body.
 ▶ Check to make sure brace is not too hot prior to application.
 ▶ Brace should be appropriately sized to the patient to ensure proper support and function.
 ▶ Care should be taken with patients with delicate or compromised skin, especially the elderly, not to apply the brace too tightly, which MAY cause pressure and shear to the fragile tissue. PATIENTS SHOULD BE INSTRUCTED ON APPROPRIATE TENSIONING DURING ONGOING USE. An under sleeve or STOCKINETTE may be used for added protection.
 ▶ Do not heat Exos™ products to greater than 212°F (100°C).
 ▶ FOR SINGLE PATIENT USE ONLY.

OPERATING BOA® LACING SYSTEM

To Loosen:
 ▶ Lift Boa® Knob. (Figure 1)
 ▶ Pull Tab. (Figure 2)

To Close and Tighten:
 ▶ Push Boa® Knob down. (Figure 3)
 ▶ Turn Boa® Knob clockwise. (Figure 3)
 ▶ Boa® Knob must be in the down locked position when wearing brace.

HEATING AND APPLICATION
 ▶ Open and unfold brace. (Figure 4)
 ▶ Proper heating temperature is 200°F (93°C).
 ▶ Heat brace in Exos™ recommended oven approximately 5 minutes or until brace is pliable. Brace should not be too hot to handle comfortably.
 ▶ Ensure that skin is completely dry before application.
 ▶ Fit brace on patient. (Figure 5)
 ▶ Remove slack in lace by tightening Boa® Knob.
Caution: Do not tighten brace fully until brace has returned to room temperature otherwise indentations in plastic may occur.
 ▶ Mold brace into desired position. (Figure 6)
 ▶ Hook-and-loop fastener tab may be secured in the desired position.
 ▶ Once brace has hardened, the Boa® Knob may be tightened to desired tension.
Warning: The brace should not be worn too tightly. This may produce excessive mechanical force or shear forces to the skin.

CLEANING INSTRUCTIONS

- Loosen or remove the brace.
- Wash inside and outside of brace and skin with antibacterial soap and water (86°F / 30°C).
- Rinse thoroughly with water.
- Use a hair dryer set on high volume and cool setting to thoroughly dry the interior of the brace, and skin.
- Retighten the brace.

For fitting tips on each Exos™ device:
 Visit the Exos™ Provider Center at:
<http://exosmedical.com/provider/SAFB-OT> or
 Scan the QR image with your smartphone

LIMITED PRODUCT WARRANTY
 Exos™ covers defects in workmanship and materials up to 8 weeks from date this product is dispensed. Damage caused by normal wear and tear, abuse, or alterations is not covered.

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