ENGLISH

Before using the device, please read the entire instruction manual. Correct application and care is vital to the proper functioning of the device.

INTENDED USE / INDICATIONS
The Exos Rhiza provides stabilization to the first metacarpophalangeal (MCP) joint and the carpometacarpal (CMC) joint. The Exos Rhiza may be used preoperative, postoperative, post-traumatic as well as for degenerative conditions.

CONTRAINDICATIONS
Patients with known allergies to foams, adhesives, or plastic products.

⚠️ WARNINGS and PRECAUTIONS
Do not heat brace on the body.
Check to make sure brace is not too hot prior to application.
Brace should be appropriately sized to the patient to ensure proper support and function.
Care should be taken with patients with delicate or compromised skin, especially the elderly, not to apply the brace too tightly, which MAY cause pressure and shear to the fragile tissue. PATIENTS SHOULD BE INSTRUCTED ON APPROPRIATE TENSIONING DURING ONGOING USE. An under sleeve or stockinette may be used for added protection.
Do not heat Exos™ products to greater than 122°F (50°C).
FOR SINGLE PATIENT USE ONLY.

OPERATING BOA® SYSTEM

To Loosen Boa®:
1. Lift Boa® Knob. (Figure 2)
2. Pull Tab. (Figure 2)

To Close and Tighten Boa®:
1. Push Boa® Knob down. (Figure 3)
2. Turn Boa® Knob clockwise. (Figure 3)
3. Boa® Knob must be in the down/locked position when wearing brace.

HEATING AND APPLICATION

1. Open and unfold brace.
2. Proper heating temperature is 200°F (93°C).
3. Heat brace in Exos™ oven approximately 3-4 minutes or until brace is pliable. Brace should not be too hot to handle comfortably.
4. Ensure that skin is completely dry before application.
5. Fit brace on patient.
6. Remove slack in lace by tightening Boa® Knob.
7. Caution: Do not tighten brace fully until brace has returned to room temperature otherwise indentations in plastic may occur.
8. Accessory strap should be secured in the proper position. (Figure 4)
9. Mold brace into desired position. (Figure 5)
10. Once brace has hardened, the Boa® Knob may be tightened to desired tension.
Warning: The brace should not be worn too tightly. This may produce excessive mechanical force or shear forces to the skin.

CLEANING INSTRUCTIONS

1. Loosen or remove the brace.
2. Wash inside and outside of brace and skin with antibacterial soap and water (86°F/30°C).
3. Rinse thoroughly with water.
4. Use a hair dryer set on high volume and cool setting to thoroughly dry the interior of the brace, and skin.
5. Retighten the brace.

For fitting tips on each Exos™ device visit the Exos™ Provider Center at:
<http://exosmedical.com/provider/Rhiza> or Scan the QR image with your smartphone

MATERIAL USED
EVA foam, spandex, nylon, terpolymer, polyester, steel, plastic, nylon 6/6 resin, Antimicrobial treatment

LIMITED PRODUCT WARRANTY
Exos™ covers defects in workmanship and materials up to 8 weeks from date this product is dispensed. Damage caused by normal wear and tear, abuse, or alterations is not covered.
To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

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