BOA® LOCKING RING


GIVE TO PATIENT

BOA® LOCKING RING
 ANILLO DE BLOQUEO BOA®
 BOA®-SICHERUNGSRING
 ANELLO DI BLOCCO BOA®
 ANNEAU DE VERROUILLAGE BOA®
 BOA® SLUITRING
 POISTNÝ KRÚŽOK BOA®
 BOA®-LÄSERINGEN
 BOA® LÄSRING
 BOA®-LUKITUSRENGAS
 POJISTNÝ KROUŽEK BOA®
 ANEL DE BLOQUEIO BOA®
 BOA®-LÄSERINGEN
 BOA® 扣环用于防止
 BOA® 固定リングは

Distributed by: DJO, LLC
 1430 Decision Street,
 Vista, CA 92081
 P 800.336.6569
 F 800.336.6569
 djo.com

no. 10139 Rev D
 USA and International Patents Pending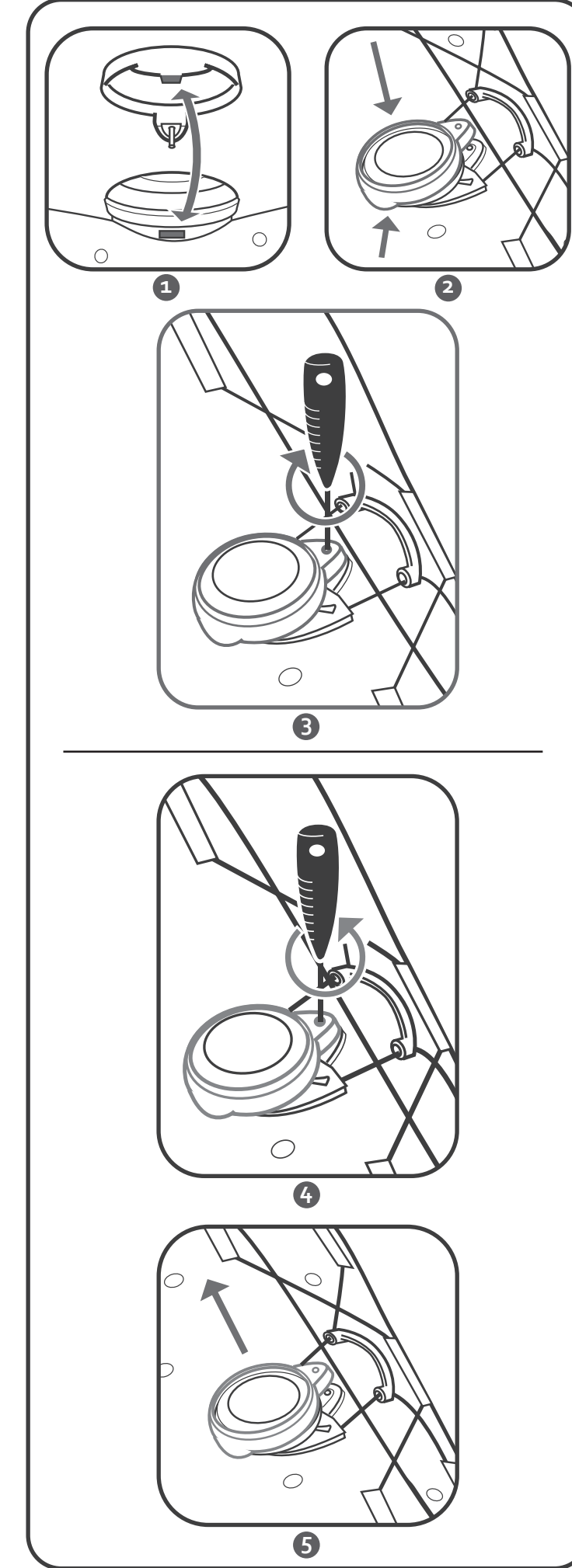

ENGLISH

Before using the device, please read the entire instruction manual. Correct application and care is vital to the proper functioning of the device.

INTENDED USE
 The Boa® Locking Ring is intended to prevent the Boa® Reel on the Exos™ Braces from being lifted to the unlocked position, thus inhibiting the use of the Boa® Reel.

⚠ WARNINGS AND PRECAUTIONS
 ▶ If the brace is applied with a Boa® Locking Ring, the patient must not get brace wet.

SECURING THE LOCKING RING
 ▶ Ensure Boa® Reel is in the down and locked position before applying Locking Ring.
 ▶ Insert hook of Locking Ring cap into the slot. (Figure 1 & 2)
Warning: If the Boa® Reel is in the up position, the Locking Ring will not engage properly.
 ▶ Use the small Boa® screwdriver key to secure the Locking Ring by turning the key clockwise until Locking Ring fully seats onto the Boa® Reel base. (Figure 3)
Caution: Do not over tighten the screw.

REMOVING THE LOCKING RING
 ▶ Loosen the screw on the Locking Ring by turning the screw driver key counterclockwise. (Figure 4)
 ▶ Lift off Locking Ring. (Figure 5)

To learn about the care of your state-of-the-art Exos™ Brace...
 Visit the Patient section of our website at:
<http://exosmedical.com/patient/LR> or
 Scan the QR image with your smartphone

LIMITED PRODUCT WARRANTY
 Exos™ covers defects in workmanship and materials up to 8 weeks from date this product is dispensed. Damage caused by normal wear and tear, abuse, or alterations is not covered.

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