# RGFB

## PATIENT INSTRUCTIONS

## RADIAL GUTTER FRACTURE BRACE

## GIVE TO PATIENT

### RADIAL GUTTER FRACTURE BRACE (RGFB)

#### ORTESIS PARA FRACTURAS RADIALES

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
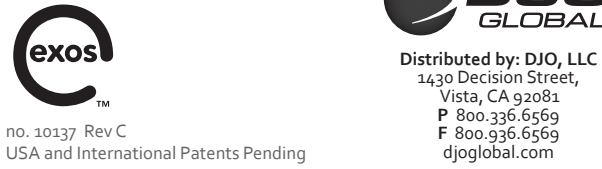
Distributed by: DJO, LLC  
1430 Decision Street,  
Vista, CA 92081  
P 800.336.6569  
F 800.936.6569  
djo.com

Exos™ Corporation  
1122 Red Fox Road  
Arden Hills, MN 55112

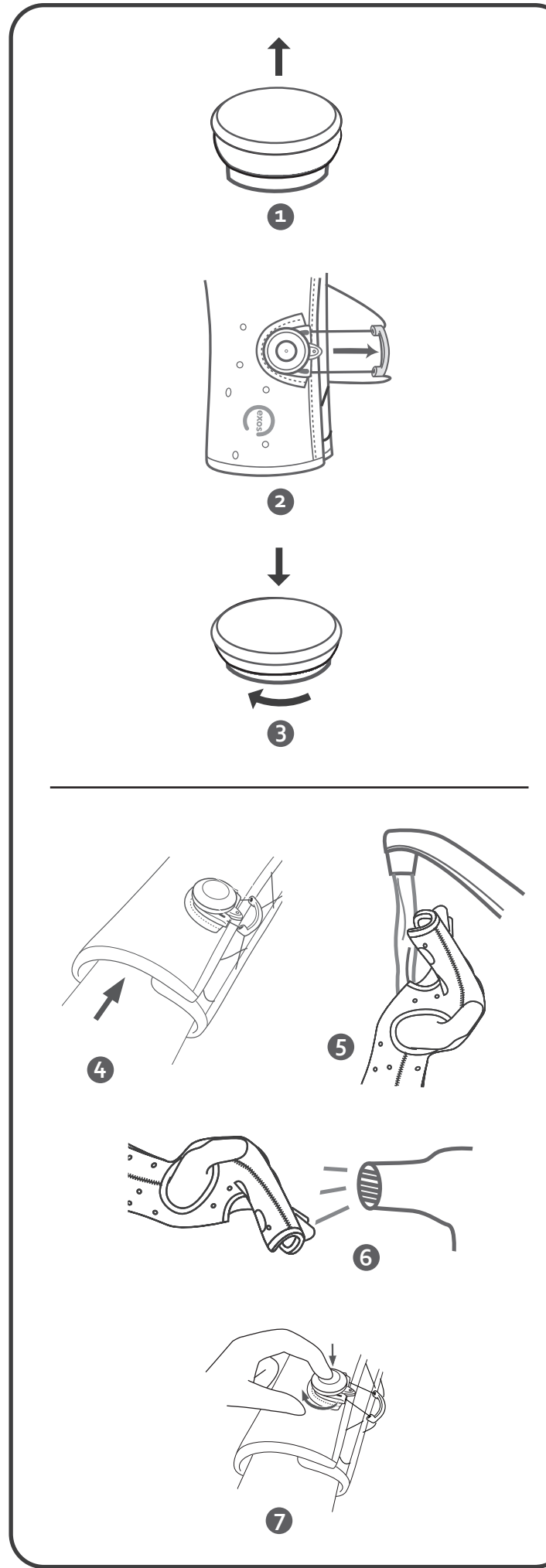Medical Device & QA Services Ltd  
Timperley, Cheshire,  
WA15 7SN United Kingdom

### ENGLISH

**INTENDED USE / INDICATIONS**  
The Radial Gutter Fracture Brace (RGFB) provides stabilization to the second and third metacarpals, second and third metacarpophalangeal joints, and the second and third digits, and the wrist joint.

**CONTRAINDICATIONS**  
Patients with known allergies to foams, adhesives, or plastic products.

**⚠️ WARNINGS AND PRECAUTIONS**  
**Instruction for wearing, opening, closing or removing the brace should be provided by a physician.**  
► Consult your physician if unusual swelling, increasing discomfort, pain, numbness or discoloration appears.  
► If brace does not fit properly, or is irritating, it may be reformed and adjusted by a healthcare professional.  
► If brace breaks or cracks, see your healthcare professional for a replacement.  
► Wearing the brace too tight may result in skin irritation, odor, or other skin issues.  
► If the brace is applied with a Boa® Locking Ring or the patient is instructed not to loosen or remove the brace, the patient must not get brace wet.  
► The brace may soften and lose its desired shape when exposed to heat over 130°F (55°C).  
► Do not expose brace to moisturizers, chemicals, or solvents, as they may affect durability.  
► FOR SINGLE PATIENT USE ONLY.

### OPERATING BOA® LACING SYSTEM

**To Loosen:**  
► Lift Boa® Knob. (Figure 1)  
► Pull Tab. (Figure 2)

**To Close and Tighten:**  
► Push Boa® Knob down. (Figure 3)  
► Turn Boa® Knob clockwise. (Figure 3)  
► Boa® Knob must be in the down locked position when wearing brace.

### 3 KEYS FOR CARE AND COMFORT

- Not Too Tight - Wear it Right and leave some "WIGGLE ROOM" (Figure 4)**  
► Exos™ Braces are meant to be worn under light pressure and not tight to the skin. A little wiggle room inside an Exos™ Brace promotes air circulation and helps maintain dry healthy skin.  
► An over-tightened brace creates "shear" or pressure on the skin during motion, and can result in irritation, rash, odor, and skin issues.  
► Wearing the brace too loose will reduce support and may compromise healing.

- If the brace and skin get wet, they "CANT STAY WET"**  
► The Brace may be washed or worn while swimming or bathing, only if the physician allows the patient to loosen or remove the brace. If not, the patient cannot get the brace and skin wet.  
► It is imperative that the inside surface of the brace and the patient's skin underneath are completely dry after swimming or bathing.  
► See below for cleaning and drying instructions.

- Keep your "SKIN AND BRACE CLEAN AND DRY"**  
Exos™ braces are treated with antimicrobial to help eliminate odor causing bacteria. However, when possible we recommend cleaning your brace.  
► If your physician has given permission to loosen or remove the brace, wash the brace (inside and out) and your skin with antibacterial soap, rinse thoroughly, and completely dry the skin and brace (inside and out) with a hair dryer.  
► A clean and dry environment inside the brace will help maintain healthy skin and reduce odor and skin issues.

- Loosen or remove the brace as instructed by physician.**
- Wash inside and outside of brace and skin with antibacterial soap and water (86°F / 30°C). (Figure 5)**
- Rinse thoroughly with water.**
- Use a hair dryer set on high volume and cool setting to thoroughly dry the interior of the brace, and skin. (Figure 6)**
- Retighten the brace. (Figure 7)**

**To learn about the care of your state-of-the-art Exos™ Brace...**  
Visit the Patient section of our website at:  
<http://exosmedical.com/patient/rgfb> or  
Scan the QR image with your smartphone

### LIMITED PRODUCT WARRANTY

Exos™ covers defects in workmanship and materials up to 8 weeks from date this product is produced. Damage caused by normal wear and tear, abuse, or alterations is not covered.

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