WB w/Boa®

PATIENT INSTRUCTIONS

WRIST BRACE WITH BOA®

GIVE TO PATIENT


WRIST BRACE WITH BOA®
ORTESIS DE MUÑECA CON BOA®
EXOS™ HANDGELENKORTHESSE OFFEN MIT BOA®
TUTORE PER POLSO CON BOA®
ATTELLE DE POIGNET AVEC BOA®
WRISTBRACE MET BOA®
ORTÉZA ZÁPÄSTIA SO SÝSTÉMOM BOA®
ZÁPÄSTNÍ ORTÉZA SE SYSTÉMOM BOA®
APOYO PARA O PULSO COM BOA®
具有 Boa® 绑带系统的腕护
Boa® 付き手首ブレ

DJO GLOBAL
Distributed by: DJO, LLC
1420 Decision Street,
Vista, CA 92081
P: 800.338.6569
F: 800.338.6569
djoglobal.com

CE **EC** **RE**
Medical Device & QA Services Ltd
Timperley, Cheshire.
WA15 7SN United Kingdom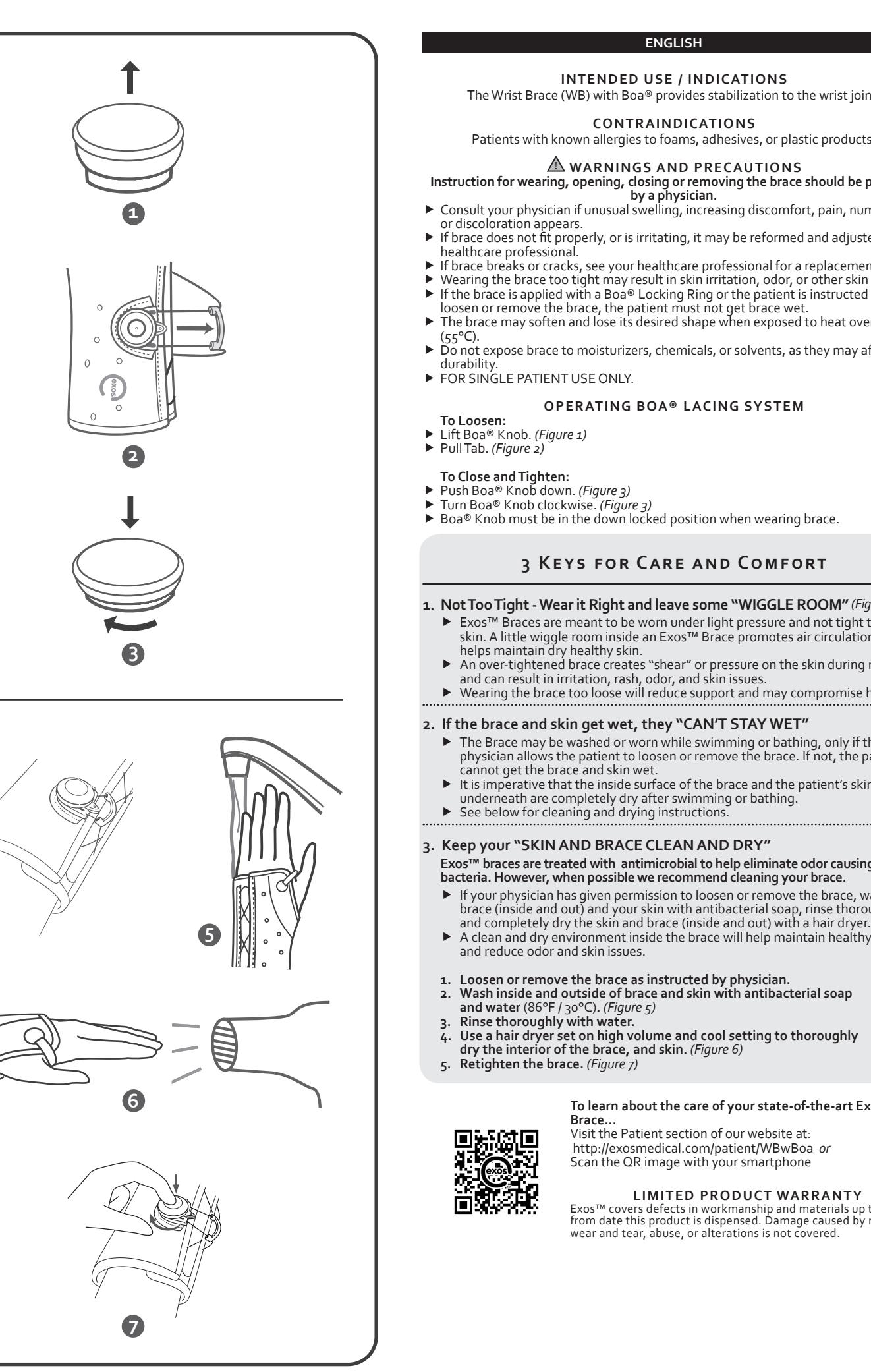

ENGLISH

INTENDED USE / INDICATIONS

The Wrist Brace (WB) with Boa® provides stabilization to the wrist joint.

CONTRAINDICATIONS

Patients with known allergies to foams, adhesives, or plastic products.

WARNINGS AND PRECAUTIONS

Instruction for wearing, opening, closing or removing the brace should be provided by a physician.

Consult your physician if unusual swelling, increasing discomfort, pain, numbness or discoloration appears.

► If brace does not fit properly, or is irritating, it may be reformed and adjusted by a healthcare professional.

► If brace breaks or cracks, see your healthcare professional for a replacement.

► Wearing the brace too tight may result in skin irritation, odor, or other skin issues.

► If the brace is applied with a Boa® Locking Ring or that patient is instructed not to loosen or remove the brace, the patient must not get brace wet.

► The brace may soften and lose its desired shape when exposed to heat over 130°F (55°C).

► Do not expose brace to moisturizers, chemicals, or solvents, as they may affect durability.

► FOR SINGLE PATIENT USE ONLY.

OPERATING BOA® LACING SYSTEM

To Loosen:

Lift Boa® Knob. (Figure 1)

Pull Tab. (Figure 2)

To Close and Tighten:

Push Boa® Knob down. (Figure 3)

Turn Boa® Knob clockwise. (Figure 3)

Boa® Knob must be in the down locked position when wearing brace.

3 KEYS FOR CARE AND COMFORT

1. Not Too Tight - Wear it Right and leave some "WIGGLE ROOM" (Figure 4)

► Exos™ Braces are meant to be worn under light pressure and not tight to the skin. A little wiggle room inside an Exos™ Brace promotes air circulation and helps maintain dry healthy skin.

► An over-tightened brace creates "shear" or pressure on the skin during motion, and can result in irritation, rash, odor, and skin issues.

► Wearing the brace too loose will support and may compromise healing.

2. If the brace and skin get wet, they "CAN'T STAY WET"

► The Brace may be washed & worn while swimming or bathing, only if the physician allows the patient to loosen or remove the brace. If not, the patient cannot get the brace wet or remove the brace.

► It is important that the inside surface of the brace and the patient's skin underneath are completely dry after swimming or bathing.

► See below for cleaning and drying instructions.

3. Keep your "SKIN AND BRACE CLEAN AND DRY"

Exos™ braces are treated with antimicrobial to help eliminate odor causing bacteria. However, when possible we recommend cleaning your brace.

► If your physician has given permission to loosen or remove the brace, wash the brace (inside and out) and your skin with antibacterial soap, rinse thoroughly, and completely dry the skin and brace (inside and out) with a hair dryer.

► A clean and dry environment inside the brace will help maintain healthy skin and reduce odor and skin issues.

1. Not too strak - Wear it right and leave some "MARGE DE MANOEUVRE" (Figure 4)

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